

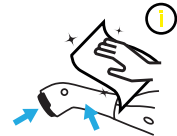



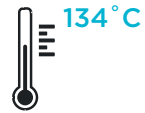



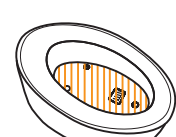
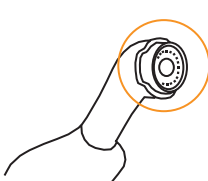
DISINFECTION AND STERILIZATION PROCEDURES

STEPS	CALIBRATION TIPS	RAYPLICKER DEVICE
- 1 - DISINFECTION	<p>AUTHORIZED WIPES (⚠)</p> <p>IN / OUT </p>	 
- 2 - RINSING	<p>RINSE THOROUGHLY WITH WATER</p> <p>IN / OUT  </p>	
- 3 - STERILIZATION	<p>AUTOClave STERILIZATION</p>  	

⚠ Disinfectant wipes for surfaces of non-invasive medical devices with **bactericidal, fungicidal, virucidal and tuberculocidal** properties. Please follow the manufacturer's instructions. (contact time required). See below for details of sensitive areas.

ⓘ This device disinfection protocol, required after each use to avoid cross-contamination, also applies to the optional use of single-use protective sheaths.

RISKY AREAS IN CASE OF EXCESSIVE USE OF DISINFECTANT CHEMICALS

DEVICE	DOCKING STATION	MEASURING HEAD
		

NON-COMPLIANT PRODUCTS AND PROCESSES

HEAVILY SOAKED WIPES	SPRAYABLE SOLUTION	DECONTAMINATION BATH	ULTRASONIC BANK
