





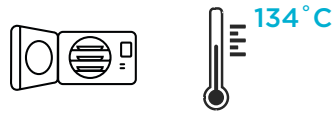


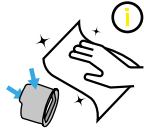


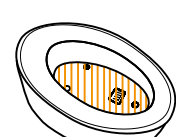
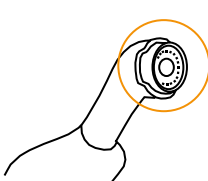
## DISINFECTION AND STERILIZATION PROCEDURES

| STEPS                  | CALIBRATION TIPS  | RAYPLICKER DEVICE  |
|------------------------|---|--|
| - 1 -<br>DISINFECTION  | <p><b>AUTHORIZED WIPES (⚠)</b></p> <p>IN / OUT </p>  |   <p>DRYING </p> |
| - 2 -<br>RINSING       | <p><b>RINSE THOROUGHLY WITH WATER</b></p> <p>IN / OUT  </p> |  |
| - 3 -<br>STERILIZATION | <p><b>AUTOClave STERILIZATION</b></p>    |  |





 Disinfectant wipes for surfaces of non-invasive medical devices with **bactericidal, fungicidal, virucidal and tuberculocidal** properties. Please follow the manufacturer's instructions. (contact time required). See below for details of sensitive areas.

 This device disinfection protocol, required after each use to avoid cross-contamination.

## RISKY AREAS IN CASE OF EXCESSIVE USE OF DISINFECTANT CHEMICALS

| DEVICE  | DOCKING STATION   | MEASURING HEAD   |
|---|---|--|
|  |  |  |

## NON-COMPLIANT PRODUCTS AND PROCESSES

| HEAVILY SOAKED WIPES  | SPRAYABLE SOLUTION  | DECONTAMINATION BATH   | ULTRASONIC BANK   |
|---|---|--|---|
|  |  |  |  |

