

## BEST PRACTICES FOR USING RAYPLICKER COBRA



- ✓ GREY EDGES PARALLEL TO SCREEN EDGES
- ✓ MOUTH OPENED AND BREATHING THROUGH THE NOSE
- ✓ SALIVA LIMITATION
- ✓ CENTERING THE TOOTH, WITHOUT ANGLE NOR INCLINATION
- ✓ TIP UNDER THE LIP
- ✓ TIP IN CONTACT WITH ADJACENT TEETH
- ✓ TIP IN CONTACT WITH THE GINGIVA
- ✓ DON'T MOVE DURING THE ACQUISITION (1s)

## INCORRECT PRACTICES

**TIP INCORRECTLY POSITIONED**



**MOUTH CLOSED**



**CONDENSATION**




**EXCESSIVE PRESSURE ON THE GINGIVA**



**TOOTH CENTERED INCORRECTLY**



**WRONG ANGLE**

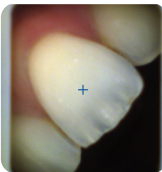

**WRONG INCLINATION**



**VISIBLE LIP**



## CALIBRATION TIPS

FAILURE TO RESPECT THE DISINFECTION AND STERILIZATION PROCEDURES CAUSES A HASTY DEGRADATION OF THE TIPS.

A DAMAGED CALIBRATION TIP (DISCOLORATION, MODIFIED ASPECT,...) MUST BE CHANGED.

LIFETIME VERIFIED FOR 100 AUTOCLAVE CYCLE PASSES.



**DAMAGED TIP**  
Internal aspect and/or external shiny and sticky



**NEW TIP**  
Internal aspect and external gray and matte