RAYPLICKER COBRA

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ZH

EN - DISINFECTION AND STERILIZATION PROCEDURES

STEPS	CALIBRATION TIPS
- 1 - DISINFECTION	<p>AUTHORIZED WIPES (⚠)</p> <p>INT / EXT</p> 
- 2 - RINSING	<p>RINSE THOROUGHLY WITH WATER</p> <p>INT / EXT</p> 
- 3 - STERILIZATION	<p>AUTOClave STERILIZATION</p>  <p>134 °C</p>

STEPS	DEVICE RAYPLICKER COBRA
DISINFECTION	<p>AUTHORIZED WIPES (⚠)</p>    <p>DRYING</p>

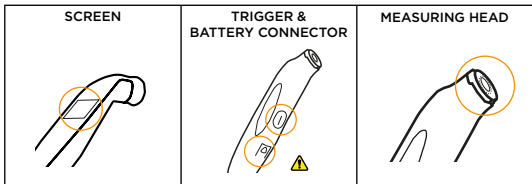
Disinfectant wipes for surfaces of non-invasive medical devices with bactericidal, fungicidal, virucidal and tuberculocidal properties. Please follow the manufacturer's instructions (required contact time).

This device disinfection protocol is mandatory after each use to avoid cross-contamination.


RISKY AREAS

IN CASE OF EXCESSIVE USE OF DISINFECTANTS

 NO CLEANING IF THE BATTERY PACK IS DISCONNECTED.

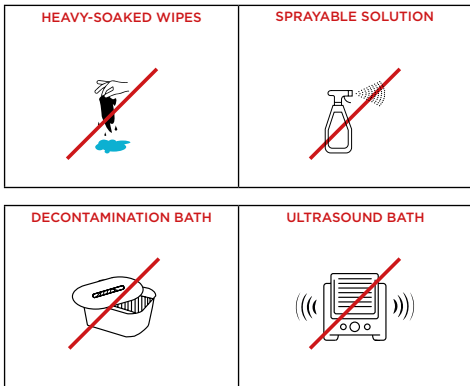NON-COMPLIANT PRODUCTS AND PROCESSES

BEST PRACTICES FOR USING RAYPLICKER COBRA

- ✓ GREY EDGES PARALLEL TO SCREEN EDGES
- ✓ MOUTH OPENED AND BREATHING THROUGH THE NOSE
- ✓ SALIVA LIMITATION
- ✓ CENTERING THE TOOTH, WITHOUT ANGLE NOR INCLINATION
- ✓ TIP UNDER THE LIP
- ✓ TIP IN CONTACT WITH ADJACENT TEETH
- ✓ TIP IN CONTACT WITH THE GINGIVA
- ✓ DON'T MOVE DURING THE ACQUISITION (1s)

INCORRECT PRACTICES

TIP INCORRECTLY POSITIONED

MOUTH CLOSED

CONDENSATION

EXCESSIVE PRESSURE ON THE GINGIVA

TOOTH CENTERED INCORRECTLY

WRONG ANGLE

WRONG INCLINATION

VISIBLE LIP

CALIBRATION TIPS

FAILURE TO RESPECT THE DISINFECTION AND STERILIZATION PROCEDURES CAUSES A HASTY DEGRADATION OF THE TIPS.

A DAMAGED CALIBRATION TIP (DISCOLORATION, MODIFIED ASPECT,...) MUST BE CHANGED.

LIFETIME VERIFIED FOR 100 AUTOCLAVE CYCLE PASSES.

DAMAGED TIP
Internal aspect and/or external shiny and sticky

NEW TIP
Gray and mat internal and external appearance

BATTERY PACK

DISCONNECTING THE BATTERY PACK IS EXCLUSIVELY RESERVED FOR MAINTENANCE OPERATIONS REQUIRING PRIOR AGREEMENT FROM BOREA. (FAILURE TO COMPLY WITH THIS RULE WILL VOID THE WARRANTY)

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