





RAYPLICKER COBRA UPDATE PROCEDURE

07.2023




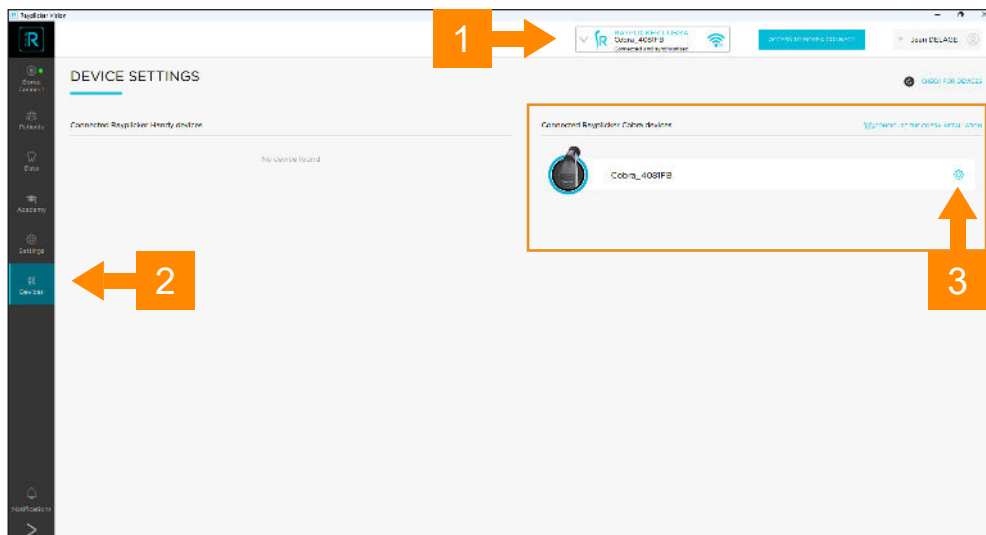

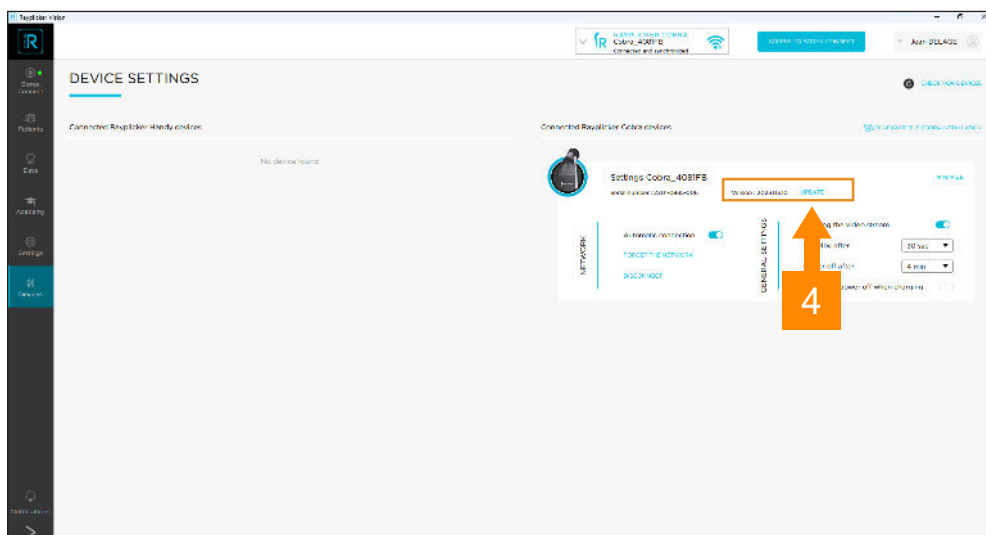

- 1 - Check that your Rayclipper Cobra is connected and synchronized
- 2 - Click on “Devices” menu
- 3 - Open the settings of your connected Cobra by clicking on the toothed wheel icon.

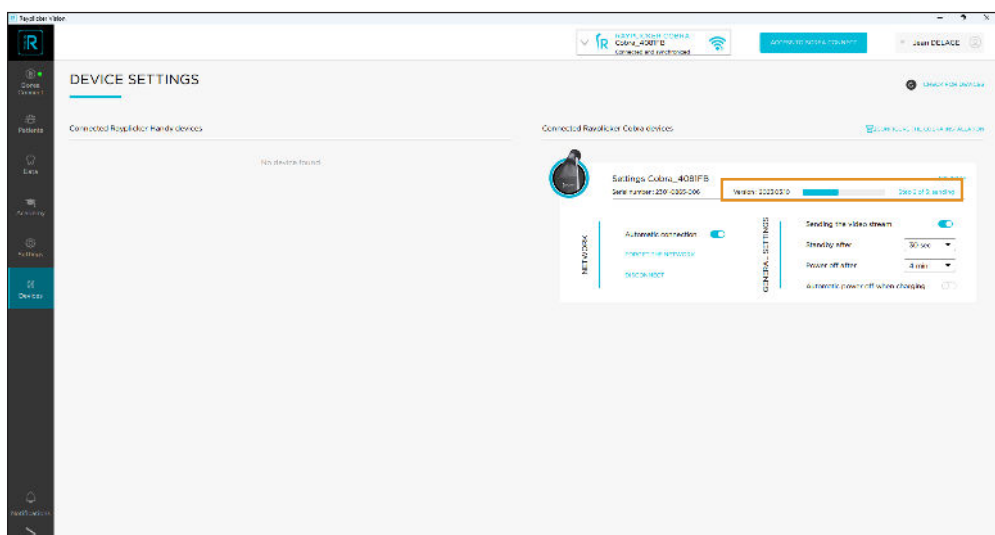

- 4 - Make sure your cobra is on the home screen “R” and if an update is available, click on “UPDATE”



- 5 - The update is done in three automatic steps.

Step 1: Download firmware to the Vision	
Step 2 - Upload the firmware to Cobra	
Step 3: The Cobra flashes the new firmware	
<div>UPDATE DONE</div>	<p>This message appears on Rayplicker vision software when update is done and successful.</p> <div> <div>Update completed, the device will reboot.</div> <div>CONTINUE</div> </div>

Rayplicker Cobra update procedure

BOREA SAS, 6 Allée Duke Ellington, 87100 Limoges Cedex, France.
+33 (0)5 44 00 01 70 - contact@borea-dental.com - www.borea-dental.com