

## DISINFECTION AND STERILIZATION PROCEDURES

STEPS	CALIBRATION TIPS	COBRA DEVICE
- 1 - DISINFECTION	<p><b>AUTHORIZED WIPES</b> (⚠)</p>	
- 2 - RINSING	<p><b>RINSE THOROUGHLY WITH WATER</b></p>	
- 3 - STERILIZATION	<p><b>AUTOClave STERILIZATION</b></p>	

⚠ Disinfectant wipes for surfaces of non-invasive medical devices with bactericidal, fungicidal, virucidal and tuberculocidal properties. Please follow the manufacturer's instructions (required contact time). See below for risk areas.  
 1 This device disinfection protocol is mandatory after each use to avoid cross-contamination.

## RISKY AREAS IN CASE OF EXCESSIVE USE OF DISINFECTANTS

<p><b>SCREEN AND CONNECTORS</b></p>	<p><b>MEASURING HEAD AND BUTTONS</b></p>	<p><b>BASE CONNECTORS</b></p>
-------------------------------------	--	-------------------------------

⚠ NO CLEANING IF THE BATTERY PACK IS DISCONNECTED.

## NON-COMPLIANT PRODUCTS AND PROCESSES

<p><b>HEAVY-SOAKED WIPES</b></p>	<p><b>SPRAYABLE SOLUTION</b></p>	<p><b>DECONTAMINATION BATH</b></p>	<p><b>ULTRASOUND BATH</b></p>
----------------------------------	----------------------------------	------------------------------------	-------------------------------